# HYDROCYN® aqua

- DE** Gebrauchsanweisung
- EN** Instructions for Use
- FR** Indications d'Utilisation
- IT** Istruzioni per l'uso

## AUF DEM ETIKETT VERWENDETE SYMBOLE / SYMBOLS USED ON LABELLING / SYMBOLES UTILISÉS SUR L'ÉTIQUETTE / SIMBOLI UTILIZZATI SULL'ETICHETTATURA

- REF** Katalognummer / Catalogue Number / N° de catalogue / Codice/Articolo
- LOT** Chargennummer / Batch Code / Code du lot / Numero di Lotti
- MD** Medizinisches Produkt / Medical Device / Dispositif médical / Dispositivo Medico
- UDI** Zeigt an, dass ein Barcode eine Unique Device Identification (UDI) enthält / Indiques a barcode as containing Unique Device Identification / Présente un code-barres qui contient le numéro d'identification unique du dispositif / Indikeran en streckkod som innehåller unik produktidentifiering

 Gebrauchsanweisung befolgen / Consult instructions for use / Consulter le mode d'emploi / Vedere le istruzioni per l'uso

 Achtung / Caution / Attention / Attenzione

 Temperaturgrenze (Lagertemperatur 4°C - 30°C) / Temperature limit (storage temperature 4°C - 30°C) / Température limite (température de stockage de 4°C à 30°C) / Limite di temperatura (conservare tra 4°C e 30°C)

 Vor Sonnenlicht schützen / Keep away from sunlight / Tenir à l'écart de la lumière du soleil / Tenere lontano dalla luce solare

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**INDICATION:** HYDROCYN aqua Gel is for moistening and debridement of acute and chronic wounds such as leg ulcers of all kinds, diabetic ulcers, surgical wounds, burns (1<sup>st</sup> and 2<sup>nd</sup> degree) and pressure ulcers.

### APPLICATION:

1. Debride the wound, if necessary and/or cleanse the wound with a HYDROCYN aqua solution. 2. Apply a generous amount of HYDROCYN aqua Gel to the entire wound bed. 3. Apply a thin coating to the periwound skin area and allow HYDROCYN aqua Gel to dry. 4. Cover with an appropriate wound dressing or covering. 5. Change the dressing once a day or as indicated by the clinical condition of the wound. Maintain a moist wound environment between dressing changes.

**STORAGE CONDITIONS:** HYDROCYN aqua Gel should be stored in a cool and dry place (recommended storage temperature is 4°C to 30°C) in its original packaging. Protect against direct sunlight and extreme weather conditions. After use, the tube must be closed again.

**SHELF LIFE:** The product is stable for 18 months if unopened and in its original packaging. Use prior to the expiration date noted on the packaging label. There is a shelf-life of 90 days after opening as long as it is closed immediately after use and protected from contamination or direct contact with patients.

**RISK OF RE-USE:** HYDROCYN aqua Gel cannot be re-used if it is dispensed out from the tube.

**ATTENTION:** Do not use the product if the tamper-proof seal is damaged or tampered with. Packaging has to be discarded if signs of contaminants are seen. Do not dilute the product or use it in combination with other creams or ointments unless instructed to do by a doctor. Keep out of the reach of children.

HYDROCYN aqua Gel may not be used in cases of hypersensitivity to any of the components of the preparation. Consult your doctor if the wound continues to deteriorate even after the use of gel. Notify your doctor in case of any adverse effect.

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**Bactiguard AB**
**OBELIS S.A**